

Yes!Star 

Yes!Star Dental X-Ray Film



Shanghai Yestar Co., Ltd



Product Series

Product Name	Type	Size	Specification	Package
Yesstar GY-D(Bright Room)	Speed D	30×40mm	100pcs/box	80box/carton
Yesstar GY-E(Dark Room)	Speed E	30×40mm	150pcs/box	80box/carton
Yesstar Monobath GYM- II	Liquid(For Bright Room)	250ml	250ml/btl	40btl/carton
Yesstar Lateral Cephalogram	Green Sensitive Film	8×10inch	50pcs/box	10box/carton
Yesstar Panoramic Film	Green Sensitive Film	6×12in	50pcs/box	10box/carton
Yesstar Dental Film Set	Speed D		50pcs bright room films 250ml monobath syring	50box/carton



Wash Dental Film In Lightroom(Speed D) *(Bright Room set)*

Wash Condition and Method

1. Film the dental film by normal procedure.
2. Extract 3-5ml monobath with injector, inject from the upper right corner into the dental film bag.
3. Press on the pinhole, shake or knead the dental bag properly to coat the monobath on the film evenly, tear off film bag after 60-90s, and rinse off with water. (Note: Do not use your fingers to wipe the film's surface in case of scratched,)
4. Dry the film off for inspection and diagnosis.



(GY-D)

Lightroom Boxset
(Bright Room set)

Exposure Conditions

Focal Distance 20cm(8in) , 65KV,1.5mA

Service Life	Anterior Teeth Exposure Time	Carine Exposure Time	Posterior Teeth Exposure Time
1-3 years	0.6s-1.0s	0.6s-1.0s	0.7s-1.2s
3-5 years	0.7s-1.2s	0.7s-1.2s	0.8s-1.5s
≥5 years	0.8s-1.4s	0.8s-1.4s	0.9s-1.5s

[This is for your reference only.]

Focal Distance 20cm(8in) , 70KV,8mA

Upper Teeth		Time	Lower Teeth		Time
321	↑ 123	0.4s	321	123	0.34s
654	456	0.54s	654	456	0.34s
87	78	0.6s	87	↓ 78	0.4s



Wash Dental Film In Lightroom(Speed D)

Attentions

1. There is a convex point on the horns of dental film bag, used to judge the left or right teeth. And the convex point side indicates the side is faced radiation source.
2. The temperature of using disposable monobath is good to set as 22°C, higher temperature makes the image too black and formed fog easily, lower makes the image too quiet. Users can adjust the temperature according to the situation until clear.
3. Should be careful to inject liquor with a syringe, avoid transpierce the dental film bag to cause leakage.
4. Need to use the assorted Yes!Star monobath. Please seal monobath after use.
5. Avoid monobath splash into eyes, rinse with water immediately if contacted.



(GY-D/GY-1)



Enterprise Introduction

Shanghai Yestar Co., Ltd

Yes!Star is Captial Group's investment company in China. Until now Yes!Star has set up 26 branches, and makes marketing network in whole country. The main business is Fuji Film image product sales and after-sale services. Yes!Star is unique general agent of Fuji Film civil image products in China. In 2011, Yes!Star follow to launch some new products, including Yes!Star brand industrial x-ray film, Yes!Star brand dental x-ray film, Fuji medical digital laser film ,etc. And then we expand to professional sensitive consumables manufacture and sales in medical and industrial area. Belong with Capital Group, Guangxi Yes!Star Co., Ltd. is a manufacturing enterprise, which owns advanced technique and equipment imported from Japanese. It adopts Fuji strict quality control system, and builds professional research and examination team. Guangxi Yes!Star Co., Ltd is worthy of the name that modernization image materials production base. At present, Yes!Star has developed to one of the leading company in domestic image industry.

Web: www.yesstarnet.com.cn

Add: 8th Floor ,Building A, Greenland Tower ,No.58 Xing Jian Dong Road, Shanghai, China

Tel: 021-64890111 Fax: 021-64890555

The distributor in Vietnam:

Viet Nha Trading and Services Company Limited (Viet Nha Co., Ltd.)

Add: 22A/3 Street #5, Binh Trung Ward, Ho Chi Minh City, Vietnam.

Tel: 028.66604836 Mobile: 0906.759.139 Web: www.vietnha.com



Pursuit of Perfect Quality